#10

COLLECTION KOLLEKTION

01.2016

MARKUS T

MARKUS T EYEGLASSES MARKUS T BRILLEN

100% HANDMADE IN GERMANY

#10

DESIGN	4
ME	8
SUN	14
TITAN	16
COLOURS	20
INFORMATION	22

100 % HANDMADE IN GERMANY MARKUS T glasses are manufactured by people with a fine touch and a love for precision – on demand – in Westphalia, the place of business. Thus, the wearer can be sure to have a pair of glasses that was custom-made for him or her.

100 % COMFORT Glasses by MARKUS T are so light and comfortable that the wearer hardly notices them. What there is to notice is the intelligent technical construction, ensuring the comfort. And there is the pure design that conveys an elegant simplicity that has delighted jurors at national and international design contests.

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D3.313

The new DESIGN/D3 – Minimum Weight, Maximum Design. The D3 is minimalistic and delicate, yet certainly anything but boring. The shapes and colours give the frames a young and modern appearance – the technical details are subtle and unbelievably simple at the same time.

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D2.060

D1.041

Less is more: The puristic, reduced glasses that look as made out of a single wire. They will be hardly noticed.

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HINGE / SCHARNIER

Hinge connection without screws,
long-lasting and low-maintenance

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MATERIAL / MATERIAL

Ultra-light TMI synthetic material
and antiallergenic titanium

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LENS MOUNTING / GLASHALTERUNG

Intelligent clip system closes the
frame without screws.

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M2.263

Smart for colours: The glasses with zeitgeist for people who want to show their colours. For their individual look – ME.

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02—
Bridge *Brücke*03 —
Height *Höhe*

M2.655

The sunglasses that combine great comfort and ideal functionality.

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LENS MOUNTING / GLASHALTERUNG

Closing system with a nylon thread,
without any screws

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T3.326

Genuine values: The glasses made of ultra-thin and ultra-resistant titanium. Pure comfort – even for people suffering from allergies.

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
	LENS RECOM- MENDATION	MIN. LENS THICKNESS	MIN. GROOVE WIDTH	RCMD. GROOVE DEPTH	TEMPLE FITTING	TEMPLE SHORTENING
						
DESIGN CLASSIC D1, D2	Plastic lenses Index 1.6	2.0 mm	1.1 mm	0.6 mm	Titan simply bend. TMi-temple: Heat a wide area at 140°C / 284°F for 60 sec. When soft bend within 10 sec.	Shortening as usual. Finish off TMi-temple with fine rubber, don't polish.
DESIGN D3	Plastic lenses Index 1.6	2.0 mm	1.0 mm	0.5 mm	Titan simply bend. TMi-temple: Heat a wide area at 140°C / 284°F for 60 sec. When soft bend within 10 sec.	Shortening as usual. Finish off TMi-temple with fine rubber, don't polish.
ME M1	Plastic lenses Index 1.6	2.0 mm	1.1 mm	0.6 mm	Heat a wide area at 140°C / 284°F for 60 sec. When soft bend within 10 sec.	Shortening as usual. Finish off TMi-temple with fine rubber, don't polish.
ME M2	Plastic lenses Index 1.6	2.0 mm	0.7 mm	0.5 mm - 0.6 mm	Heat a wide area at 140°C / 284°F for 60 sec. When soft bend within 10 sec.	Shortening as usual. Finish off TMi-temple with fine rubber, don't polish.
ME M3	Plastic lenses Index 1.6	2.0 mm	0.7 mm	0.5 mm - 0.6 mm	Heat a wide area at 140°C / 284°F for 60 sec. When soft bend within 10 sec.	Shortening as usual. Finish off TMi-temple with fine rubber, don't polish.
TITAN T2	Plastic lenses Index 1.6	2.0 mm	0.9 mm	0.6 mm	Titan: Simple adjustment as plug-in temple or anatomic adjustment on the ear. TMi: Heat a wide area at 140°C / 284°F for 60 sec. When soft bend within 10 sec.	Easy shortening of 1 cm: pull off temple tip → bend and break off the temple at the groove → cut off the silicone temple tip at the mark → put some lubricant on temple → push the temple tip back on.
TITAN T3	Plastic lenses Index 1.6	2.0 mm	0.7 mm	0.6 mm	Simple adjustment as plug-in temple or anatomic adjustment on the ear.	Easy shortening of 1 cm: pull off temple tip → bend and break off the temple at the groove → cut off the silicone temple tip at the mark → put some lubricant on temple → push the temple tip back on.

CLEANING ADVICE

Cleaning with clear water without additives.

Don't use products with solvents or alcohol.

In order to clean the TMi-temple never use spirit but acetone instead.

Avoid cleaning the fram in the ultrasonic bath.

MORE INFORMATION

Detailed information on the collections on the sheet "Indications and Information".

COLLECTION KOLLEKTION

01.2016

Markus Temming GmbH
Isselhorster Straße 374
33334 Gütersloh, Germany

T +49 (0)5241. 74343-0
F +49 (0)5241. 74343-10

info@markus-t.com
markus-t.com